

POCT WAIVED TESTS AND QUALITY CONTROLS

****All Testing Materials And Controls Must Be Dated And initialed Upon Opening****			
Test	QC Frequency	Slides/Test strips /Cuvette/devices Exp.	Controls Exp. Date
Multistix/Uristix	When first opened	Up to date of exp. on bottle	Chek-Stix - up to date of expiration on bottle
ColoScreen Slides Colo. Developer	When first opened When first opened	Up to date of exp. on slide Up to date of exp. on vial	
Gastrocult slides Gast. Developer	When first opened When first opened	Up to date of expiration on slide Up to date of expiration on vial	
IM Confirms II Pregnancy devices	When first opened	Up to date of expiration on device	
Controls			90 days after opening **Refrig**
Glucometer	Each day of use Each new vial of test strips Change Code Chip each time a new box of test strips is opened		
Test Strips		Up to date of expiration on vial	
Controls			90 days after opening
HemoCue	2 Levels each day of use 2 levels each new bottle of microcuvettes Control cuvette QC each day of use when applicable		
Microcuvette		90 days after opening	
Liquid Controls			30 days after opening**Rm Temp or Refrig.**

POCT WAIVED TESTS AND QUALITY CONTROLS

****All Testing Materials And Controls Must Be Dated And initialed Upon Opening****			
Test	QC Frequency	Slides/Test strips /Cuvette/devices Exp.	Controls Exp. Date
DCA Vantage	2 Levels each day of use 2 Levels on each New lot 2 Levels when each new box of cartridges is opened Optics check every 3 months and/or after maintenance		
Reagent Calibration Card	Upon opening a new box of reagents Prior to running qc	Up to date of exp on box ** Refrig.** 90 days if kept at Rm Temp.	
Cartridge			
Controls			90 days after reconstitution ** Refrig.**
Cholestech	2 Levels each day of use. 2 Levels on each new shipment 2 Levels on each new Lot 2 Levels on each new box		
Optic Check	Each day of use After analyzer is serviced or moved.		
Cassettes		Up to date of exp on box ** Refrig.** 30 days** Rm Temp.**	
Controls			30 days after opening **Refrig**
pHydrion Paper	**QC done by Clinical Laboratories ***	N/A	
Signify™ Strep A test	External QC – Upon opening a new kit, a new bottle of reagent, or a new canister of dipsticks Internal Proc. QC – First sample tested each day of patient testing.	Reagents and Liquid controls - up to date of expiration on box Test dipsticks – 90 days after initial opening date	

POCT WAIVED TESTS AND QUALITY CONTROLS

****All Testing Materials And Controls Must Be Dated And initialed Upon Opening****			
Test	QC Frequency	Slides/Test strips /Cuvette/devices Exp.	Controls Exp. Date
QuickVue[®] Influenza Test	External QC – upon opening a new kit Internal Proc. QC – First sample tested each day of patient testing	Up to date of expiration on box	
BINAX NOW•RSV Test	External QC – upon opening a new kit Internal Proc. QC – First sample tested each day of patient testing	Up to date of expiration on box	
Coaguchek	2 levels of EQC (electronic QC) each day of use 2 levels each day of use 2 levels upon opening new box of test strips Change Code Chip each time a new box of test strips is opened		
Test Strips		Up to date of exp. on box if kept in ref. (2-8 °C) 60 days if stored at Rm Temp.	
Controls			Up to date of exp. If kept in Ref. (2-8 °C) 30 days if stored at Rm Temp.